

CPAP Cleaner S600

User Manual



Before operation, please read this user manual carefully and follow the instructions to operate properly. Thank you for purchasing S600 CPAP cleaner, which is designed to effectively kill germs and bacteria with O3. You can use it to sanitize CPAP machine, mask and tubing completely. There are 3 working modes for your different needs. The S600 sterilizer makes the entire sanitization process becomes carefree, requiring no water and other chemical cleaning.


Safety Precautions:

- Keep this product out of the reach of children and do not let them play with it.
- Do not stare directly at UV light or expose your skin directly under UV light.
- Do not disassemble or repair the product, in case of causing damage to the product.
- Keep the product away from water or other liquid.
- Please use the original power adapter and USB cable, in case of the abnormality of this product.

Kindly Reminder:

Before using this cleaner, please make sure all packaging materials are removed and no visible damage to the cleaner and accessories.

PRODUCT DESCRIPTION

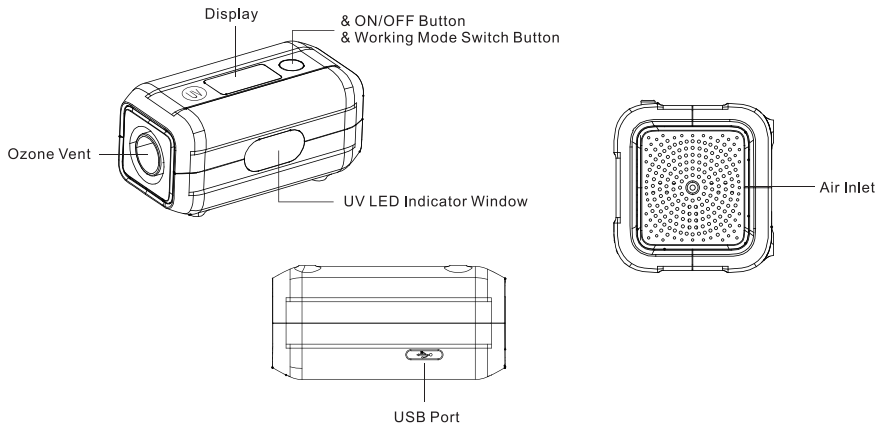

PACKAGE CONTENTS

- 1.S600x1
- 2.A/C Chargerx1
- 3.USB Cablesx1
- 4.Tube Connectorx2
- 5.Travel Bagx1
- 6.Flexible Hosex1
- 7.User Manualx1
- 8.Carbon Filter Cottonx5
- 9.Sealed Zip Bag



S600



Power Adapter




USB Cable



Tube Connector



Travel Bag

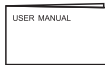

Manual



Flexible Hose



Activated Carbon Filter Cotton



Sealed Zip Bag

LED UV DISINFECTION -- CREATIVE TECHNOLOGY

Equipped inside this S600 cleaner, the high-intensive LED UV disinfection light sterilizes coming air previously; sterilized fresh air is then converted to Ozone; fresh air together with ozone is then transported to CPAP tubing, CPAP machine and masks to have them sterilized. LED UV lights can be turned on/off by UV button.

CLEAN CPAP MACHINE

1. For the CPAP machine configured with humidifier, drain water in reservoir before starting the sanitization process.
2. Connect CPAP machine and S600 with end of the tubing from your CPAP mask; long press power button for 2 seconds to turn on the cleaner with "beep" and screen will be lighted on.


3. It goes to defaulted 30 minutes mode which includes 20 minutes ozone disinfection and 10 minutes air blowing to empty the residue ozone. After 30 minutes working, the cleaner will turn off automatically with "beep" and screen will also be off.
4. To stop disinfection during working, long press power button for 2 seconds and it will turn off with "beep".

Tips: Please use your CPAP machine 30~60 minutes later after disinfection, ensuring ozone has dissipated completely.



CLEAN CPAP MASK AND TUBING


- Put CPAP mask in the travel bag and connect the bag with S600 cleaner by tubing. Long press power button for 2 seconds to turn on the cleaner with “beep” and enters to defaulted 30 minutes mode with screen lighted on. Short press power button to switch to 20 minutes mode, which includes 10 minutes ozone disinfection and 10 minutes air blowing to empty the residue ozone. After 20 minutes working, the cleaner will turn off automatically with “beep” and screen will also be off.
- To stop disinfection during working mode, long press power button for 2 seconds and it will turn off with “beep”.
- Please use your CPAP mask and tubing 30~60 minutes later after disinfection, ensuring ozone has dissipated completely.

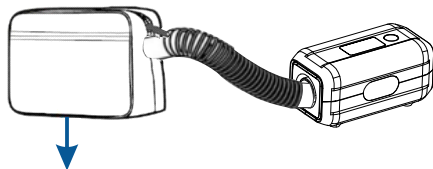

Mask inside

CLEAN CPAP MACHINE MASK AND TUBING

- For the CPAP machine configured with humidifier, drain water in reservoir before starting the sanitization process.
- Put your mask in the travel bag. Connect the three-way connector to S600 cleaner. Connect the CPAP machine and the larger port of the three-way connector by tubing; connect the bag and the smaller port of the three-way connector by flexible hose. Please see below photo.
- Long press power button for 2 seconds to turn on the cleaner with “beep” and enter to defaulted 30 minutes mode with screen lighted on. Short press power button twice to switch to 40 minutes mode, which includes 25 minutes ozone disinfection and 15 minutes air blowing to empty the residue ozone. After 40 minutes working, the cleaner will turn off automatically with “beep” and screen will also be off.
- To stop disinfection during working mode, long press power button for 2 seconds and it will turn off with “beep”.
- Please use your CPAP mask and tubing 30~60 minutes later after disinfection, ensuring ozone has dissipated completely.

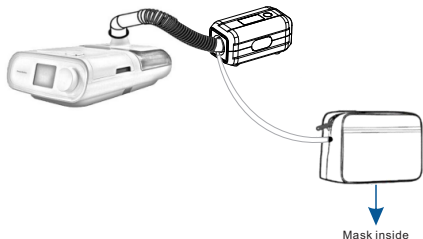

CHARGING THE BATTERY

- The S600 cleaner has a built-in battery. Please charge it when it is in low power with A/C adapter and USB cable in the package.

Charge instructions:

The battery signal on LED display flashes while charging.


Indication light is green when charging completed.

- When it is in low power, it will beep every 10 seconds for reminding and turn off after 5 times' reminding.
- If in low power, you can connect A/C adapter and USB cable to the cleaner to disinfect directly. After disinfection, please charge the battery in off mode and it takes around 4 hours to fully charge.

REPLACING ACTIVATED CARBON FILTER COTTON STEPS

1. Remove the screw with a screwdriver;
2. Remove the filter cover;
3. Replace the old filter cotton with new filter cotton;
4. Reassemble the filter cover and the screw.

Tips: The filter cotton is recommended to be replaced every 2 months

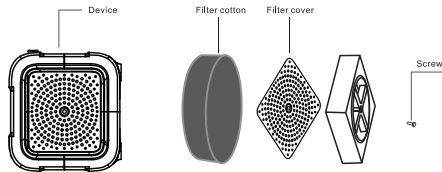

LIMITED WARRANTY

We offer a 1 year warranty on the product commencing on the date of purchase. For related accessories warranty period is 90 days.

The warranty does not cover damage due to improper use.

The warranty only covers defects in the product resulting from faults in material or workmanship, free of charge either by repairing or replacing in the complete product .

SPECIFICATIONS

Size	11.7*6.1*6CM
Weight	725g
Air flow	5.06CFM
Power consumption	5W
Power supply	DC 5V
Store temperature	-20 ~+55°C
Operation temperature	-5 ~+45°C
*S600 Capacity	2500mAh

ROUTINE CARE

1. When not using the product for a long time, please put it in the packaging box.
2. Do not place it in the damp or in the sun for a long time.
3. When using the product to sterilize, please put it in a dry, clean and ventilated environment.
4. Do not crack or strongly extrude the product, in case of shocking shell fragments and the internal precise electronic devices.
5. Do not use volatile liquids or organic liquids to wipe the product, or it may damage the surface of the product.

TROUBLES SHOOTING

Problem	Possible Solutions
The S600 does not work after turning on	Connect the S600 to the charger. Plug the charger to the power source. Turn on the S600
No flashing light or beeping sound	Make sure the charger connection is correct. If the problem is still not resolved, please send it to be repaired or replaced.
the S600 doesn't stop	Power up again. If it still has this problem, send it to be repaired or replaced.
No O ₃ generation after turning on	Power the S600 again. If it still has this problem, send it to be repaired or replaced.

keep Health Always Around You!